

SDR® flow+ Bulk Fill Flowable


PART OF THE CLASS II TOTAL PRACTICE SOLUTION™

The Unmatched Bulk Fill Flowable


SDR flow+ material is a one-of-a-kind breakthrough in dentistry: a bulk fill flowable that combines exclusive self-leveling properties and category best shrinkage stress at depths up to 4mm for excellent cavity adaptation.

Adaptation

Studies indicate that marginal gaps can lead to the development of wall lesions and can result in secondary caries¹, ultimately putting restoration longevity at risk.

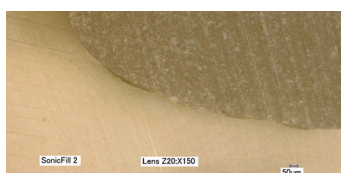

Internal microscopy image of typical cross sections of a conventional bulk fill composite in cavity preparations of extracted teeth, showing gaps being created within the adhesive layer.

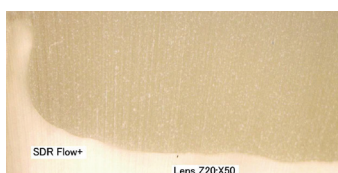

Internal microscopy image of typical cross sections of SDR® flow+ material in cavity preparations of extracted teeth, showing no margin gaps.

Efficiency

SDR flow+ material can be applied in 4mm increments without an additional liner. It self-levels and does not require additional manipulation to adapt, saving 40% in time over classic laying technique which can significantly impact office revenue.^{2,3}

TIP

Gradually withdraw the tip as the proximal box is filled. Avoid lifting the tip out of material while dispensing to minimize air entrapment. Allow the material to self-level for a few seconds, eliminating the need for further manipulation with hand instruments.

Want to learn more?


Visit www.class2restorations.com/sdr-flow-plus or contact your Dentsply Sirona Restorative specialist.



Low Shrinkage Stress

SDR™ technology enables the formation of a more relaxed polymer network, minimizing the build-up of stress. This minimizes the risk of gap formation through stress during polymerization and ensures excellent adaptation even in 4mm bulk placement.

Shrinkage stress: Shown as a ratio of SDR flow+ material vs. competitors³

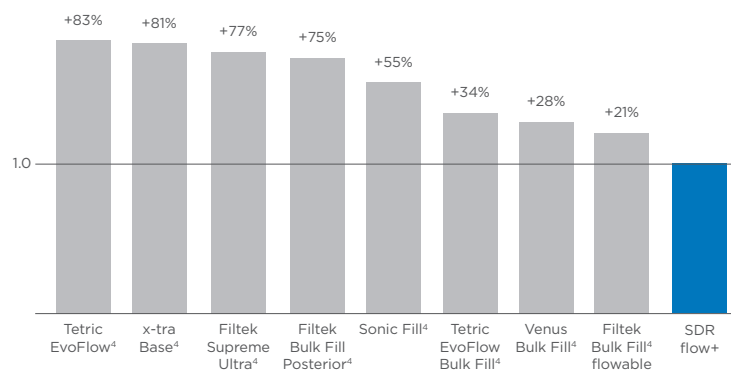

Over 85% of doctors claim SDR® flow+ material has simplified their Class I and II restorative procedures.⁵

- Exclusive self-leveling handling
- Easy placement and no manipulation
- Patented Compula® Tips delivery for easy access and visualization
- Radiopaque for x-rays



SDR® flow+ Bulk Fill Flowable

Ordering Information



Compula Tips



Syringes

SDR flow+ Introductory Kit

25 Compula Tips (0.25g each) -
Universal Shade Compule Tips gun

61C100

SDR flow+ Syringe Refill Universal Shade

2x Syringe Universal shade (1g each)
15x Syringe tips

61C130

Compula Tips Refill Universal Shade

15 Compula Tips Universal (0.25g each)

61C101

SDR flow+ Syringe Tip Refill

60x Syringe tips

61C111

A1 Shade

61C106

A2 Shade

61C107

A3 Shade

61C108

SDR flow+ Compula Tips Bulk Refill

50 Compula Tips (0.25g each)

61C103